



**motion**  
INFORMATICS

# Setup Guide

# Stimel-03




# Quick Guide for setting up the Stimel-03



1 Open the box encasing the Stimel-03.

**Do not throw away** the inside filling as it is biodegradable and compostable and can be reused

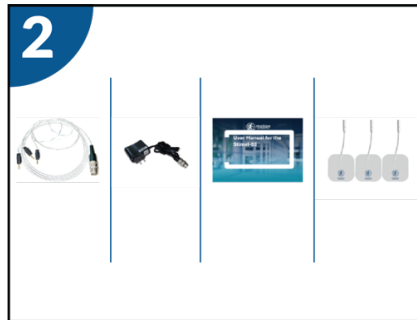

2 **Contained in the box are the items listed below**


- Electrode cable
- Main Stimel-03 Unit
- AC/DC Battery Charger (110/220 VAC; 12V DC)



- 3
- Connect the Charger to the main box by affixing the metal connector, securing it firmly.
  - Proceed to charge the until it reaches full battery capacity.
  - A full battery state is reached when a green charge light comes on instead of the red light on the plug.




4 Detach the Charger and attach the Electrode cable to the main box unit by connecting the metal connector and securely tightening it.

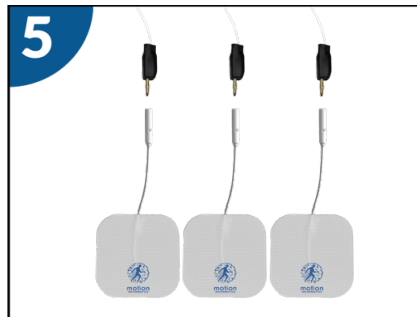

5 Connect the electrode cables to the respective cables originating from the Electrode cable.



6 Gently peel off the pad's covering, taking care as the pad has an adhesive surface.

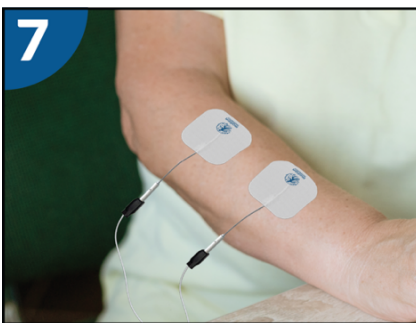

7 Fasten 2 electrode pads to upper or lower limb of the patient, and the third pad to the opposite side.



Before you start using the Stimel-03, thoroughly read and understand the User Manual. It's crucial for safe and effective operation.